




REUSABLE AND WASHABLE MASK made of 5168-NEO fabric with BLOOD BARRIER treatment // TECHNOLOGY C6 // APPLY BC with static permanent microbial agent. **AITEX APPROVED FABRIC WITH OEKO-TEX 100 CERTIFICATE.**

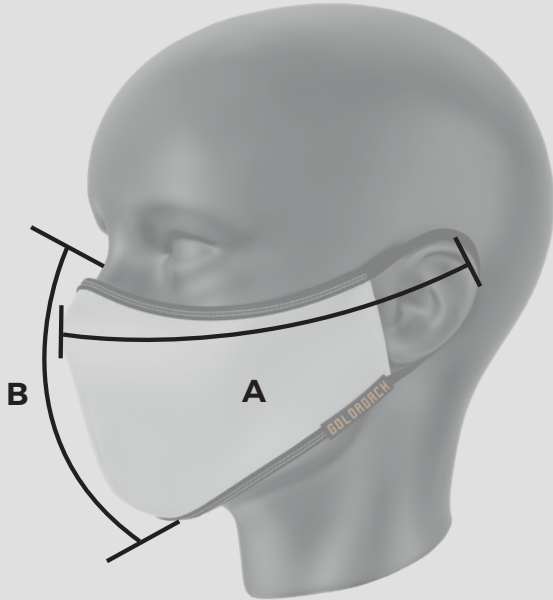

SIZES (CM)

	A	B
XXXS	14	10
XXS	15	10
XS	16	11
S	17	12
M	18	13
L	19	14
XL	20	15

A: Nose tip to upper ear fold. - **B:** Nose bridge (glasses support) to chin base.

REUSABLE AND WASHABLE MASK - LAYERS (BASIC MASK (LAYER 1) - ULTRA MASK (LAYERS 1 / 2 / 3))

LAYER 1. 5168-NEO fabric formed by two layers assembled in its bond, with a weight of 280gr / m2 and a composition of 92% pes, 8% ea.

BLOOD BARRIER formulations:

- **APPLIGUARD DPS:** Composite Fluorocarbon based on C6 technology for fluid repellency.
- **APPLIFRSH BC:** Static permanent microbial agent (does not migrate)..
- **RETICULANTES BI-25:** Helps the fluorocarbon stay longer on the fiber.
- **ANTIESTATICO AF:** Eliminates static electricity and thus prevents the adhesion of particles.


THE FABRICS AND / OR NON-FABRICS TREATED WITH THIS TYPE OF PRODUCTS PROVIDE BREATHABILITY, HELPING THE BODY TEMPERATURE TO REMAIN THE SAME AFTER PROLONGED USE, AVOIDING EXCESSIVE SWEATING.

LAYER 2. TNT NON-FABRIC 100% Non-irritating Poplpropylene (ISO 10993.3). certified under the EU 10/2011 standard with a result of 1.5 mg / dm2.

It does not contain any substance present in the High Risk Substances List, with a threshold of 0.1% according to regulation 1907/2006 / EC. NWSP 070.1 air permeability value. and tested with code 01A whose result shows that it is not cytotoxic (ISO 10993.5).

LAYER 3. "GRID" FABRIC 100% Polyester of 120gr / m2. With 100% breathability and as an internal support for TNT Filter.

THE FORMULATIONS FOR OBTAINING FLUID-REPELLENT AND ANTIBACTERIAL FINISHES ("BLOOD BARRIER") BEGIN TO DETERIORATE FROM 25 WASHES AT 40OC (HL40OC) OR 5 WASHES AT 60OC (HL60OC).

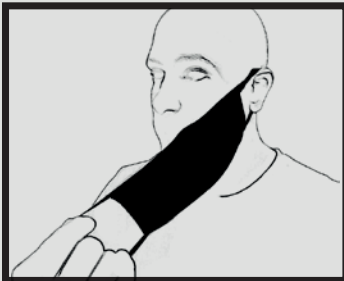

STEP 1
With the nose flap pointing up, place one ear through slit.



STEP 2
Wrap the mask around the nose and mouth.



STEP 3
Place the other ear through the slit on the other side. Mask should fit snugly on your face.



STEP 4
Adjusts the mask position to completely cover the area specified above.

LEGAL DISCLAIMER

"WARNING: THIS DEVICE IS NOT A MEDICAL DEVICE WITHIN THE MEANING OF DIRECTIVE 93/42 OR REGULATION EU / 2017/745, NOR IS IT A PERSONAL PROTECTIVE EQUIPMENT WITHIN THE MEANING OF REGULATION EU / 2016/425";