



# GOLDROACH

PROTECTS FROM THE INSIDE OUT

**REUSABLE AND WASHABLE HYGIENIC MASK** made of 5168-neo fabric with BLOOD BARRIER treatment // TECHNOLOGY C6 // APPLY BC with static permanent microbial agent. **CERTIFICATIONS ACCORDING TO SPECIFICATIONS UNE 0065: 2020 AND AITEX APPROVED FABRIC WITH OEKO-TEX 100 CERTIFICATE.**

## REUSABLE AND WASHABLE HYGIENIC MASK LAYERS

**LAYER 1 - 5168-NEO** fabric formed by two layers assembled in its bond, with a weight of 280gr / m2 and a composition of 92% PES, 8% EA.

### BLOOD BARRIER formulations:

- **APPLIGUARD DPS:** Composite Fluorocarbon based on C6 technology for fluid repellency,
- **APPLIFRSH BC:** Static permanent microbial agent (does not migrate).
- **BI-25 CROSSLINKERS:** Helps the fluorocarbon stay longer on the fiber.
- **ANTI-STATIC AF:** Eliminates static electricity and thus prevents the adhesion of particles.


**THE FABRICS AND / OR NON-FABRICS TREATED WITH THIS TYPE OF PRODUCTS PROVIDE BREATHABILITY, HELPING THE BODY TEMPERATURE TO REMAIN THE SAME AFTER PROLONGED USE, AVOIDING EXCESSIVE SWEATING.**

**LAYER 2. TNT NON-FABRIC** 100% Non-irritating Poplipropylene (ISO 10993.3), certified under the EU 10/2011 standard with a result of 1.5 mg / dm2.

It does not contain any substance present in the High Risk Substances List, with a threshold of 0.1% according to regulation 1907/2006 / EC. NWSP 070.1 air permeability value, and tested with code 01A whose result shows that it is not cytotoxic (ISO 10993.5).

**LAYER 3. "GRID" FABRIC** 100% Polyester of 120gr / m2. With 100% breathability and as an internal support for TNT Filter.


## BASIC MASK (LAYER 1) // ULTRA MASK (LAYER 1 / 2 / 3)

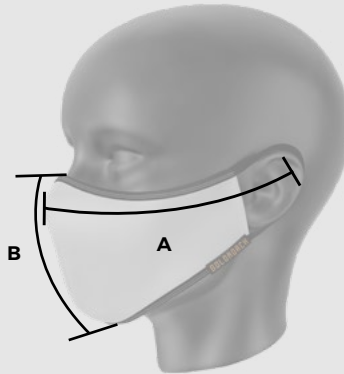


### TALLAS (CM)

	A	B
<b>XXXS</b>	14	10
<b>XXS</b>	15	10
<b>XS</b>	16	11
<b>S</b>	17	12
<b>M</b>	18	13
<b>L</b>	19	14
<b>XL</b>	20	15

**THE FORMULATIONS FOR OBTAINING FLUID-REPELLENT AND ANTIBACTERIAL FINISHES ("BLOOD BARRIER") BEGIN TO DETERIORATE FROM 25 WASHES AT 40°C (HL40°C) OR 5 WASHES AT 60°C (HL60°C).**

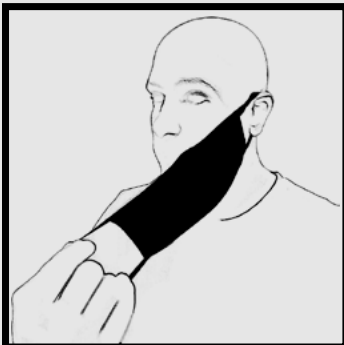


**STEP 1**  
With the nose flap pointing up, place one ear through slit.

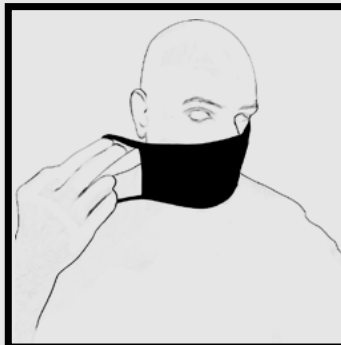

**STEP 2**  
Wrap the mask around the nose and mouth.



**STEP 3**  
Place the other ear through the slit on the other side. Mask should fit snugly on your face.

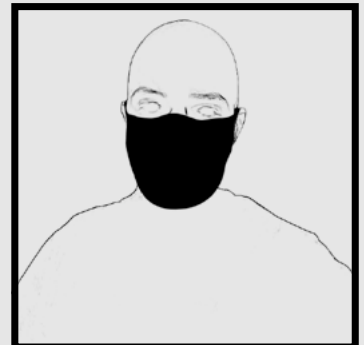

**STEP 4**  
Adjusts the mask position to completely cover the area specified above.

### LEGAL DISCLAIMER

**"WARNING: THIS DEVICE IS NOT A MEDICAL DEVICE WITHIN THE MEANING OF DIRECTIVE 93/42 OR REGULATION EU / 2017/745, NOR IS IT A PERSONAL PROTECTIVE EQUIPMENT WITHIN THE MEANING OF REGULATION EU / 2016/425";**